

## SAFEDON HOLDER INSTALLATION INSTRUCTIONS



### Step 1:

- Glue gun
- Glue (Tec7 or Superfix)




### Step 2:

- Open the tube by cutting of the seal in the front of the tube



### Step 3:

- Place the tube into the glue gun

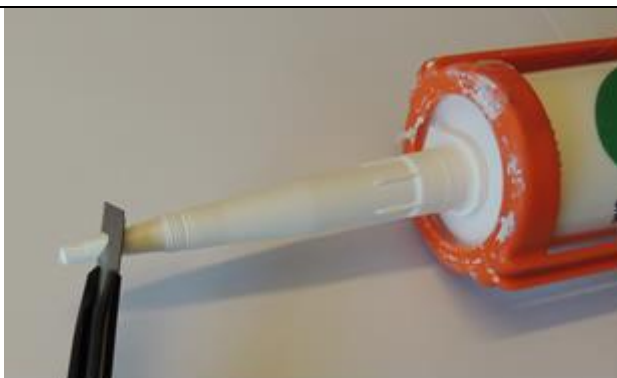

### Step 4:

- Open the tube by cutting of the tip of the funnel in front of the tube



### Step 5:

- Open single/triple SafeDon holder carton with appropriate knife



### Step 6:

- Separate the two adhesive strips from the zip bag



### Step 7:

- Remove protective paper on one side of adhesive strips



### Step 8:

- Place adhesive strips on the back of the single / triple SafeDon holder
- **DO NOT PLACE ADHESIVE STRIPS OVER KEYHOLES IN HOLDER!**



### Step 9:

- Remove protective paper on adhesive strips



### Step 10:

- Use glue gun to apply glue between the adhesive strips



### Step 11:

- Holder should be mounted approx 165 cm from ground level
- Push the single / triple SafeDon holder onto the wall and apply pressure from both hands 5-10 sec
- **LET THE GLUE HARDEN FOR 24 HOURS!**



### Step 12:

- Place SafeDon glove dispenser in holder before removing tab
- SafeDon Hygiene System is now ready to be used