

## HOW TO INSTALL

# SafeDon™

## Hygiene System

### HOLDERS

24

Wait 24 hours  
before using  
the dispenser

165 cm  
from the  
ground



## RECOMMENDED EQUIPMENT

Glue Gun



Glue: Tec 7 or Superfix




## GLUE GUN INSTRUCTIONS



### STEP 1

**Open the tube**  
by cutting of the  
seal in the front  
of the tube

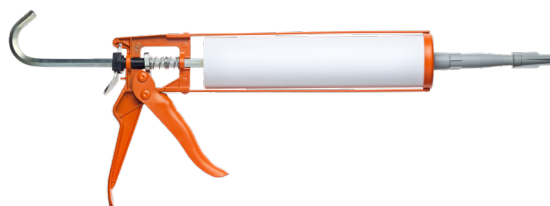

### STEP 2

**Place the tube**  
into the glue gun



### STEP 3

**Open the tube**  
by cutting of the tip  
of the funnel in front  
of the tube



[www.safedonhygienesystem.no](http://www.safedonhygienesystem.no)  
[post@safedon.no](mailto:post@safedon.no)

Norsk importør av  
SafeDon Hygiene System

DMedical  
[www.dmedical.no](http://www.dmedical.no)

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


## HOW TO INSTALL


# SafeDon™

## Hygiene System



### Holders



**Separate** the two adhesive strips from the zip bag and remove the protective paper

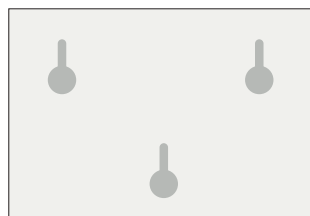


**Place the strips** On the back of the single/triple SafeDon Holder

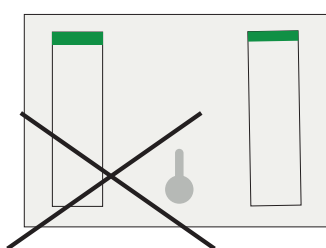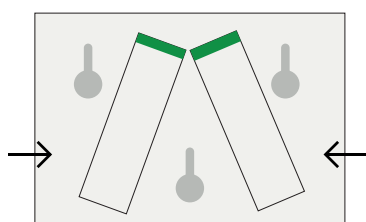


### NB!


Do not place the adhesive strips over the keyholes

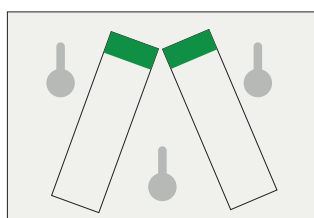

**Remove** the protective paper from the adhesive clips

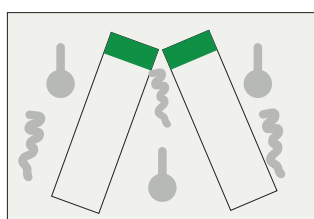

**Use the glue gun** to apply glue between the adhesive strips

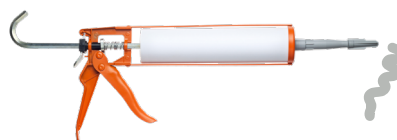

165 cm  
from the  
ground

**Push the single / triple** SafeDon holder onto the wall and apply pressure from both hands **5-10 sec.**

Wait **24 hours** before using the dispenser



The SafeDon Holders are delivered with:



**1 holder**  
(single or triple)



**2 strips**

Installed  
**165 cm**  
from the  
ground



Wait **24 hours**  
before using  
the dispenser

Place SafeDon glove dispenser in holder **before** removing tab



**SafeDon Hygiene System is now ready to be used**

